## INSTRUCTIONS FOR USE - ENGLISH

This *Instructions For Use* manual is the most comprehensive source of information for the safe and effective use of your product. Keep and consult this reference manual during the life of the product.

### IMPORTANT INFORMATION

The words WARNING, CAUTION, and NOTE have special meaning and should be reviewed.

<b>WARNING</b>	A <b>WARNING</b> highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
<b>CAUTION</b>	A <b>CAUTION</b> highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
<b>NOTE</b>	A <b>NOTE</b> supplements and/or clarifies procedural information.

### COMPLIANCE STATEMENT

Medical Device acc. Directive 93/42/EEC

This device is subject to the conformity assessment procedure according to annex VII of Directive 93/42/EEC.

### INDICATION FOR USE

This battery holster is a component of the ViVi® System. Worn on the body under sterile protection gowns, it is used by healthcare personnel to connect the battery with the helmet and by doing so, to ensure low voltage DC power supply to the helmet.

### CONTRAINDICATIONS

None known.

### FOR USE WITH

Insert REF 60402 Battery Pack HW only. Connect with REF 80100 ViVi® Helmet or REF 80120 ViVi® Helmet w/ Headlight.

### USER/ PATIENT SAFETY

#### WARNINGS:

- DO NOT open or modify this battery holster.
- DO NOT use this battery holster if damage is apparent.

#### CAUTION:

- DO NOT sterilize.
- DO NOT overtighten the connection with the battery holster when connecting the power cord of the helmet with the battery holster.
- DO NOT use tools (such as pliers) to disconnect the helmet power cord from the battery holster.

### SYMBOL DEFINITIONS

The symbols located on the equipment and/or labelling are defined in this section.

SYMBOL	DEFINITION
	Consult instructions for use
	Mark of conformity to applicable European Directives
	Manufacturer
	Serial number
	Quantity
	Article number
	Non sterile
	Dispose acc. to WEEE directive (EU) or acc. local guidelines

### HOW TO USE

#### CONNECT THE HELMET

1. Connect the power cord of the helmet to the battery holster.
2. Secure the power cord by tightening the locknut on the power cord connector. **CAUTION:** DO NOT overtighten the locknut.

#### CONNECT & DISCONNECT THE BATTERY

1. Insert the battery with connectors on battery and holster facing the same direction. **NOTE:** Geometry will prevent you from inserting the battery in any wrong direction. You will feel a distinctive lock when electrical connection are established.
2. To disconnect, gently pull back the flexible retaining bracket on the battery holster and remove the battery. **CAUTION:** do not overstretch the flexible retaining bracket.

Use the belt clip to attach the battery holster to your clothing or use pockets to secure the holster on your body.

### TECHNICAL SPECIFICATIONS

REF (Type)	60403
Housing Material	PA-12
Dimensions (LxWxH)	121 x 83 x 38 mm
Weight	58g

### CLEANING

1. Unplug the battery holster from the helmet.
2. Wipe the battery holster with a soft cloth dampened with a non-abrasive hospital disinfectant. Follow applicable hospital protocol for cleaning.
3. Dry with a lint-free towel or medical-grade compressed air.

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