# BATTERY CHARGER

# INSTRUCTIONS FOR USE

**REF** 60500 BATTERY CHARGER

ENGLISH (EN) - DEUTSCH (DE) - ITALIANO (IT)

## INSTRUCTIONS FOR USE - ENGLISH

### IMPORTANT INFORMATION

The words **WARNING**, **CAUTION**, and **NOTE** have special meaning and should be reviewed.

<b>WARNING</b>	A <b>WARNING</b> highlights a safety-related issue. <b>ALWAYS</b> comply with this information to prevent patient and/or healthcare staff injury.
<b>CAUTION</b>	A <b>CAUTION</b> highlights a product reliability issue. <b>ALWAYS</b> comply with this information to prevent product damage.
<b>NOTE</b>	A <b>NOTE</b> supplements and/or clarifies procedural information.

### COMPLIANCE STATEMENTS

This battery charger REF 60500 has been tested and has met the requirements of the following applicable regulations:

- EN 60601-1-2:2007
- FCC Title 47 CFR, Part 15, Class B/ IECS-003, IV edition
- ROHS Directive 2002/95/EC

### INDICATIONS FOR USE

The Battery Charger REF 60500 is designed to recharge the battery pack REF 60401.

### FOR USE WITH

REF 60401 Battery Pack.

### SYMBOL DEFINITIONS

These symbols are located on the device and/or labels and are defined as follows:

SYMBOL	DEFINITION
	Caution, consult accompanying documents
	Waste Electrical and Electronic Equipment Directive (WEEE)
	Recyclable Product
	Direct Current (DC)
	CE Mark
<b>LOT</b>	Serial Number
<b>P/N</b>	Catalog number
	Manufacturer

### TECHNICAL SPECIFICATION

MODEL	REF 60500
Description	Battery Charger
European Conformity	CE
Adaptor Input	100 – 240V (AC), 50–60 Hz, 1.7 A, 70W
Charger Input	24 V (DC), 2.9 A
Charging bays	4
Weight	• 390 g (13.8 oz) Battery Charger • 375 g (13.3 oz) Power Adaptor
Dimensions	200 x 130 x 42 mm (7.87 x 5.12 x 1.65 in)
Ingress Protection (IP)	IPX0
Mode of operation	Continuous

### LIMITATIONS OF USE

#### WARNINGS:

- Operation temperature for charger and power supply: -10 °C to +50 °C ( 14 °F to 122 °F)
- Upon delivery and before each use, inspect the charger and power adaptor to check for any eventual damage. DO NOT use the devices if the cord and/or plug appear damaged by the presence of cuts, holes or bent electrical contacts or cracks.
- DO NOT expose the charger and power adaptor to heat, fire, shock or high humidity conditions that can generate condensation.
- DO NOT expose the charger and power adaptor to sources of heat or sunlight. Place the battery charger in a cool place away from external heat sources.
- DO NOT use this equipment in the presence of mixtures containing a flammable anaesthetic substances including air, oxygen or nitrous oxide.
- Unless otherwise indicated, do not sterilize or immerse the charger or power adaptor in water, solvents, lubricants or other cleansing chemicals, nor expose them to them.
- DO NOT allow water to reach the charging bays on the top of the charger.
- DO NOT immerse water inside the power adaptor connector at the back of the charger. Keep the device clean and dry.
- DO NOT short-circuit, crush, open, tear, incinerate, or disassemble the charger or power adaptor.
- DO NOT modify or replace the power cord of the charger.
- DO NOT use any other power adaptor than the one included in the OXYGEN – Battery Charger in combination with the charger.
- Use the battery charger and power adaptor only for the intended applications.
- When using medical equipment such as the charger and power adaptor, follow the electromagnetic compatibility (CEM) procedures provided by the hospital.
- DO NOT use the charger by applying electrical voltages other than those indicated on the unit and in the instructions for use.

### BATTERY CHARGING

For detailed charging instructions, consult the Instructions for Use of the **OXYGEN** and **Vivi®** Helmet.

#### WARNINGS:

- Use this equipment to charge and to store REF 60401 Battery Pack only.

### CLEANING

#### WARNINGS:

- DO NOT immerse the charger and power adaptor in liquids.
- DO NOT sterilize the charger and power adaptor.

### CAUTIONS:

- DO NOT use solvents, lubricants or other chemicals, unless otherwise specified.
- DO NOT allow liquids or moisture into the electrical connections.
- DO NOT allow debris to enter electrical connections.

### TO CLEAN THE BATTERY CHARGER

1. Unplug the charger from the power outlet.
2. Wipe the battery charger with a soft cloth dampened with a non-abrasive hospital disinfectant. Follow applicable hospital protocol for cleaning.
3. Dry the equipment with a lint-free towel or medical-grade compressed air.

### STORAGE AND TRANSPORT

Recommended storage and transport conditions: -20 °C to +60 °C (-4 °F to +140 °F), up to 75 % relative humidity.

### CAUTIONS:

- Carry or store the battery charger and power adaptor in low dust environments.
- DO NOT expose the battery charger and power adaptor to extreme temperature changes. **DISPOSAL/RECYCLING**

#### WARNINGS:

• ALWAYS follow the current local recommendations and/or regulations governing environmental protection and risks associated with recycling or disposing of the equipment at the end of its useful life.

- Per the European Union (EU) Waste Electronics and Electrical Equipment (WEEE) Directive 2002/96/EC, product must be collected separately.
- DO NOT dispose of as unsorted municipal waste. Contact your local distributor for disposal information.
- Contains no hazardous substances, per RoHS standard.

## TROUBLESHOOTING

PROBLEM	CAUSE	ACTION
Battery not charging	No power	Check power indications on charger and power adaptor. Make sure the power adaptor is connected to the AC outlet and the charger.
	Incorrect battery placement	Make sure the battery is well-seated in the charging bay.
	Battery defective	Replace battery*.
	Battery charger or power adaptor defective	Replace equipment.
'Error' indication at individual charging bay (red LED)	Battery 'overcharged'	Discharge/charge the battery once.
	Defective battery	Replace battery*.
* <b>NOTE:</b> Consult the Instruction for Use of REF 60401 Battery Pack how to dispose a battery.		

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Total Healthcare Innovation GmbH
**THI - Total Healthcare Innovation GmbH**